

Mobile Solutions for Critical Situations

Cadaver Rack Assembly Instructions

• Step One



Stand unit vertically and remove security strap.

• Step Two



Remove side pin.

• Step Three



Extend one end of rack completely.

• Step Four



Insert locking pin into top support.

• Step Five



Insert locking pin into side support arm. Repeat steps 2-5 for the other half of rack. Make sure that all pins are secured.

• Step Six



Remove security straps from roller sets.

• Step Seven



Insert short rollers on bottom tier.

• Step Eight



Insert long rollers on remaining tiers. Lightly tap rollers into position.

• Step Nine



Make sure that the stops are all located on the same end. Rack is assembled and ready for use.

*Patent Pending.